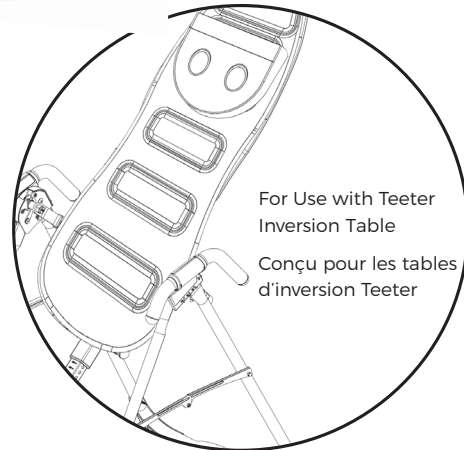
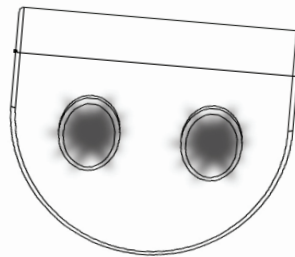
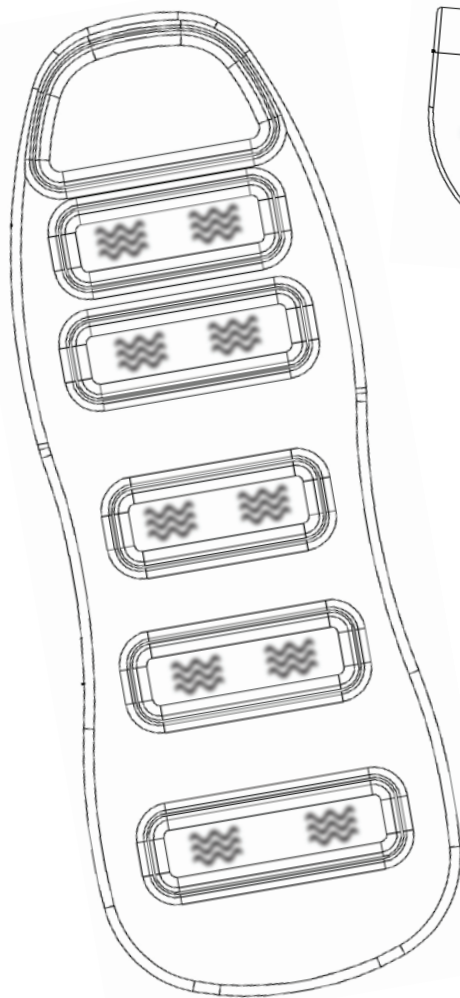
Better Back™ Vibration Cushion with Neck Arch Support

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* Specifications may vary from this image and are subject to change without notice. For use with Teeter Inversion Tables. Contents of package include Vibration Cushion and components only.

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To download and print Teeter's Better Back™ Vibration Cushion instructions, visit the product support page at teeter.com.

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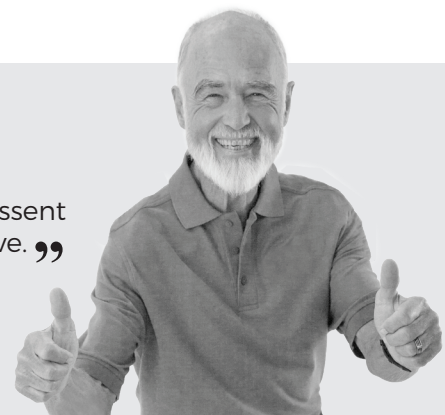
“ I created Teeter so people could live healthier, more active lives. ”

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Congratulations on your purchase of a Teeter Better Back™ Vibration Cushion! This product has been engineered with 10 powerful vibration motors and light-heat technology in the neck arch support to provide you the ultimate in relaxation and stress relief while inverted on your Teeter.

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In order to utilize this product to its fullest extent, it is critical that you read and fully understand the User Instructions prior to using your new Teeter Vibration Cushion.

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To register your product warranty, go to **teeter.com/Support/Warranty-Registration**

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If you have any questions concerning assembly or if any parts are missing, **DO NOT RETURN THE ITEM TO THE STORE OR CONTACT THE RETAILER.** Our dedicated customer service experts can help! Contact Teeter Customer Service at **800.847.0143**, or via online forms or Live Chat at **teeter.com**.

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Assembly & Use Instructions

Important Safety Instructions	1
Items for Assembly	2
Safety Warning Label & Product Specifications	3
Assembly Instructions	4-7
Use, Storage & Maintenance Instructions	8
Warranty Terms & Registration	9

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BEFORE YOU BEGIN: Review all steps before beginning use and read all precautions before using the Vibration Cushion. Carefully adhere to the Warnings and User Instructions to help ensure safety and product integrity.

IMPORTANT SAFETY INSTRUCTIONS
READ ALL INSTRUCTIONS BEFORE USING THE VIBRATION CUSHION

⚠ WARNING

FAILURE TO FOLLOW INSTRUCTIONS AND WARNINGS COULD RESULT IN SERIOUS INJURY OR DEATH.

To reduce the risk of injury:

- Read and understand all the instructions and inspect the appliance before use. When using electrical products, especially when children are present, basic safety precautions should always be followed. It is your responsibility to familiarize yourself with the proper use of this appliance and the inherent risks if these instructions are not followed, such as burns, electrocution, fire or injury. It is the responsibility of the owner to ensure that all users of the product are fully informed about the proper use of the appliance and all safety precautions.
- **ALWAYS** read, understand, and follow the instructions and warnings associated with and accompanying any supplementary equipment, such as the Teeter Inversion Table. Use on the inversion table will affect the balance, causing the table to rotate faster. Adjust your user settings per the Inversion Table Owner's Manual.
- **DO NOT** use this product as a substitute for medical attention. Consult your doctor before using the Vibration Cushion if you have any concerns regarding your health. Individuals with pacemakers should consult a doctor before use.
- **ALWAYS** unplug from the electrical outlet immediately after using and before cleaning, or before putting on or taking off parts or attachments. **NEVER** leave unattended when plugged in. To disconnect, turn all controls to the "off" position, then remove plug from outlet. **NEVER** operate if cord or plug is damaged or if it has been dropped in water.
- **ALWAYS** inspect the appliance and examine the covering before each use. Discontinue use if the covering shows any sign of deterioration, such as blistering or cracking.
- **KEEP DRY - DO NOT** operate in a wet or moist condition. **DO NOT** use outdoors.
- **DO NOT** allow the appliance to come into contact or become immersed in water or any other liquid. **DO NOT** place or store where it can fall or be pulled into a tub or sink. **DO NOT** reach for if fallen into water. Unplug it immediately.
- **DO NOT** use on insensitive skin areas or on a person with poor blood circulation. **DO NOT** use this product directly on swollen or inflamed areas or skin eruptions. Use heated surfaces carefully. Check the skin in contact with the heated area of the appliance frequently to reduce the risk of burn or injury.
- **NEVER** use while sleeping or fall asleep while using the appliance.
- **NEVER** use pins or other metallic fasteners with this appliance.
- **DO NOT** operate under a blanket or pillow. Excessive heating may occur and cause fire, electrocution or injury to persons.
- **DO NOT** allow children to use the Vibration Cushion. This appliance is not intended for use by persons with reduced physical, sensory or mental capabilities, unless they are given supervision and instruction concerning use of the appliance by a person responsible for their safety.
- **DO NOT** use attachments not recommended by the manufacturer. Use only for its intended use as described in these instructions.
- **DO NOT** carry by supply cord or use cord as handle. Keep cord away from heated surfaces.
- **DO NOT** operate where aerosol (spray) products are being used or where oxygen is being administered.
- Do not use for more than 15 continuous minutes. Extensive use could lead to excessive heating and shorter product life. Should this occur, discontinue use and allow the Vibration Cushion to cool before operating.
- **DO NOT** crush. Avoid sharp folds.
- **DO NOT** use in any commercial, rental or institutional setting. This product is intended for indoor, home-use only.
- **DO NOT** operate equipment while under the influence of drugs, alcohol, or medication that may cause drowsiness or disorientation.
- **ALWAYS** replace defective components immediately and/or keep out of use until repair. **DO NOT** attempt to repair Vibration Cushion - there are no user serviceable parts. Contact Teeter Customer Support for service.
- Refer to additional warning notices posted on the equipment. If a product label should become lost, damaged or illegible, contact Customer Service for replacement.


SAVE THESE INSTRUCTIONS

Items for Assembly


Items not shown to scale.

ITEM NO.	ITEM NAME
EP1350	Better Back™ Vibration Cushion
F51140	Neck Arch Support
F51144	Controller
F51145	AC Adapter
F51143	Controller Pouch

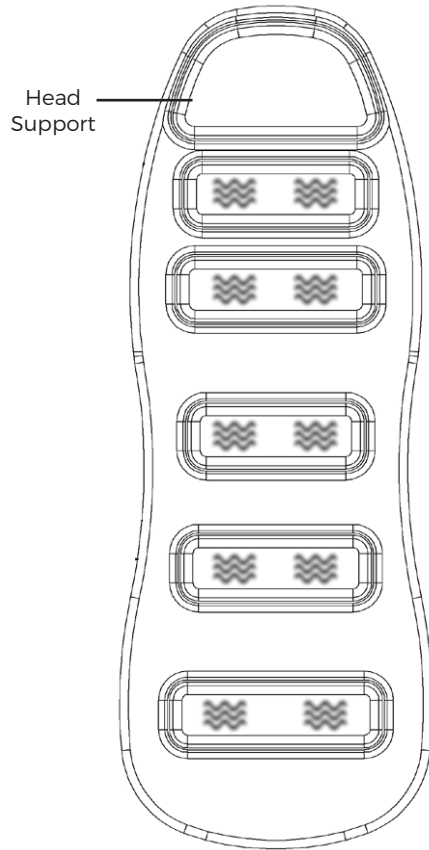
Better Back™ Vibration Cushion
with 10 vibrating motors
(EP1350)


Controller
(F51144)

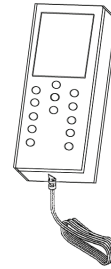AC Adapter
(F51145)

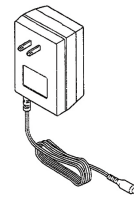Neck Arch Support
(F51140)

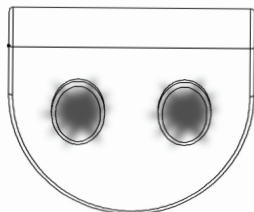Controller Pouch
(F51143)


Safety Warning Labels & Product Specifications

Important: Please review all labels and supporting materials before using your vibration cushion.

This drawing indicates the locations of the warning labels found on your product. If a label is missing, illegible or is removed, contact Teeter Customer Service to request a complimentary replacement label.

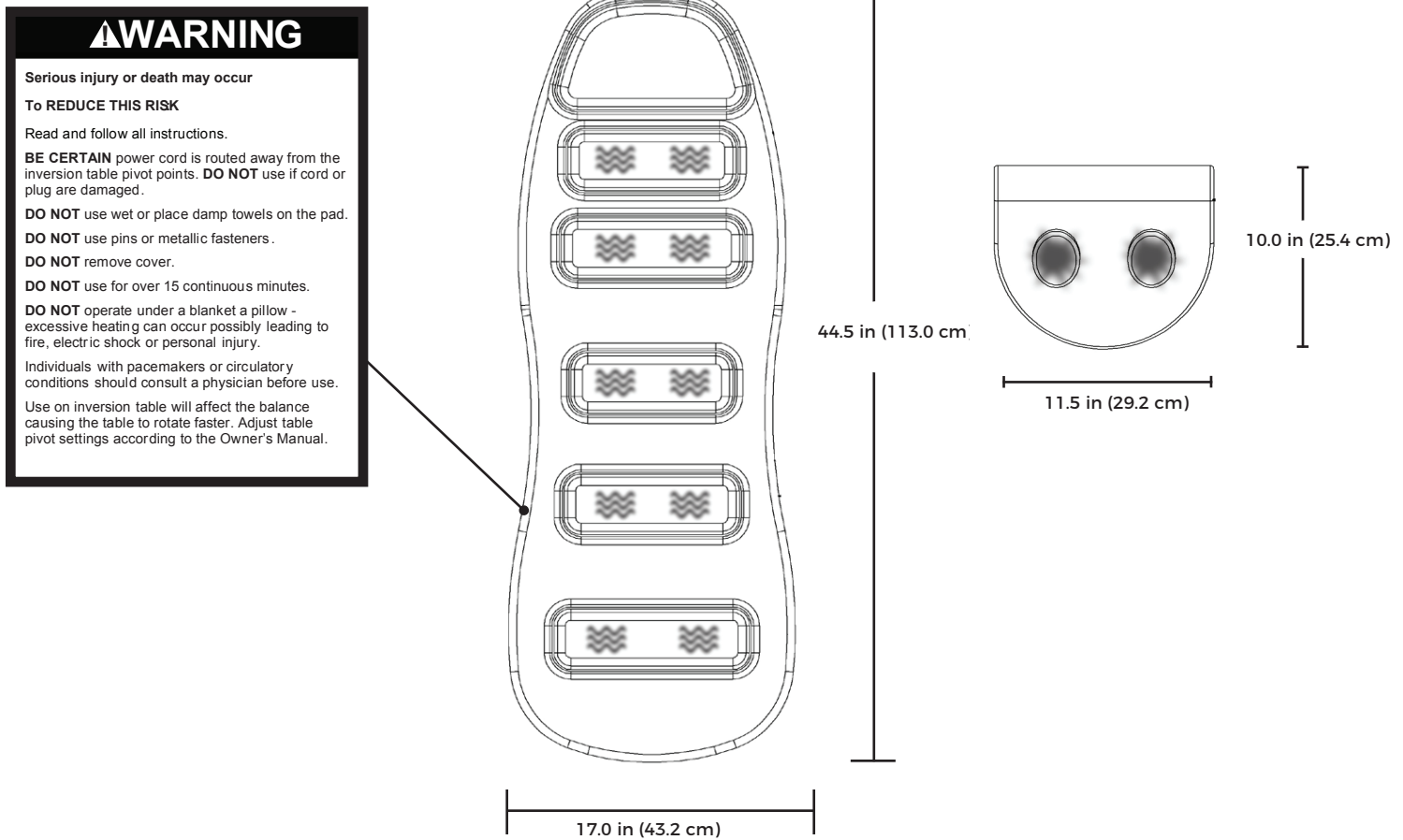Note: Image and labels below not shown at actual size.

Vibration Cushion Assembled Dimensions: 44.5 (L) x 17.0 in (W) (113.0 x 43.2 cm)

Neck Arch Support Dimensions: 10.0 (L) x 11.5 in (W) (25.4 x 29.2 cm)

Weight (approx.): 5.5 lbs (2.5 kg)

STEP ①

Attach Vibration Cushion to Table Bed

FIGURE 1

- Pull apart the Velcro on all of the straps on the Vibration Cushion.
- Lay the Vibration Cushion on top of the inversion table bed with the Head Support located at the top end of the table bed. Ensure the electrical cords are located on the left as you stand facing the unit and that they are free from the underside of the Vibration Cushion.


CAUTION: Keep your face and eyes clear of the elastic straps when stretching them for installation.

- Wrap the top horizontal straps around the table bed and secure together using the Velcro strips (Figure 1).
- Pull the top vertical strap over the top of the table bed and wrap around secured horizontal straps (Figure 2) to secure onto itself with the Velcro strips.
- Wrap the lower horizontal straps around the table bed and under the Main Shaft Housing. Secure together using the Velcro Strips (Figure 3).
- Pull the lower vertical strap under the bottom of the table bed and wrap around the secured horizontal straps to secure onto itself with the Velcro strips (Figure 3).

FIGURE 2

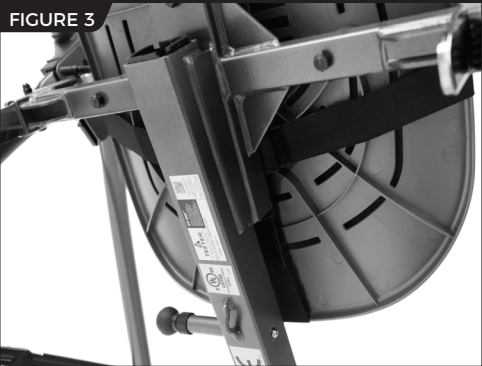
FIGURE 3

STEP ②

Attach Neck Arch to Vibration Cushion

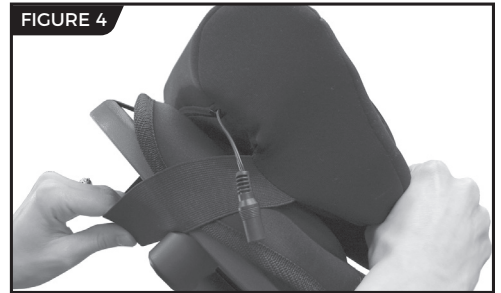- Position the Neck Arch Support on top of the Vibration Cushion so that the rounded arch is at the top of the table bed. Stretch the two elastic straps around the table bed and secure together with the Velcro Strips (Figure 4).
- The strap and arch support position can be adjusted at the time of use. The rounded top of the Neck Arch Support should be positioned so the arch supports the natural curve of your neck, and your head should rest on the Vibration cushion, not the Neck Arch Support. (Figure 5).
- Connect the Neck Arch Support cord to the input jack at the top of the Vibration Cushion (Figure 6).

STEP ③

Attach Controller

- Attach the Controller Pouch to the A-frame of the inversion table by securing the Velcro straps around the rear leg above and below the spreader arm (Figure 7).
- Connect the Controller cord to the Controller and place the Controller in the pouch (Figure 8).

STEP 4

Connect Adapter

FIGURE 9

- Secure the controller cord and electrical cord from the Vibration Cushion to the rear leg of the A-frame using the Velcro strap provided (Figure 9).
- Run the electrical cord through the secured Velcro straps of the Controller Pouch - this will help keep the cord out of the way during operation of the inversion table (Figure 10).

CAUTION: Be certain that all cords are AWAY and CLEAR of the inversion table pivot points.

- Connect the electrical cord with the AC Adapter cord (Figure 10), then plug the adapter into a 120 volt AC outlet.

FIGURE 10

Use, Storage, & Maintenance

Operating Controller

FOR LIGHT-HEAT TECHNOLOGY:

- Press the "heat" button to turn the Neck Arch Support heat lights on and off.

FOR VIBRATION:

- Press the "power" button on the Controller to start the vibrating motors.
- Press the "function" button to select vibration modes: tapping, up & down, wave, all motors, and random.
- The vibration area can be selected by pressing the buttons "neck", "upper back", "lower back", "lumbar" and "thighs".
- Press the "timer" button to select the amount of vibration time.
- Press the "fast" or "slow" buttons to adjust the speed of the vibration motors.
- Press the "high" or "low" buttons to adjust the desired intensity.

⚠ WARNING

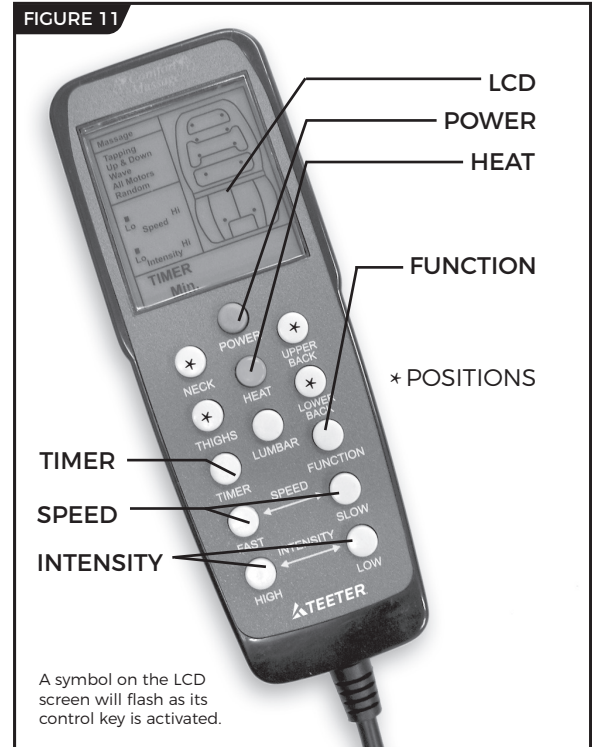
ALWAYS read, understand, and follow the instructions and warnings associated with and accompanying any supplementary equipment, such as the Teeter Inversion Table. Use on the inversion table will affect the balance, causing the table to rotate faster. Adjust your user settings per the Inversion Table Owner's Manual.

Maintenance

- Be sure to unplug the Vibration Cushion and allow it to cool prior to cleaning. Using a soft, slightly damp sponge, wipe only the surface. **DO NOT** allow the Vibration Cushion to contact or become immersed in water or any other liquid. **NEVER** clean the Vibration Cushion with brushes or abrasive cleaners, flammable fuels, glass/furniture polish or paint thinner.

Storage

- Place the Vibration Cushion in its original box or in a cool, dry place. Use caution when storing so as not to damage the internal components. Take care to avoid contact with sharp or pointed objects that might puncture the fabric surface.
- Do not wrap the power cord around the unit or attempt to hang the Vibration Cushion by the Controller cord.

The Teeter warranty set forth below and on Teeter's website applies to US and Canadian customers only. For international customers, please consult your local distributor for warranty information which will vary depending on country.

During the period starting with the day of retail purchase and continuing for one (1) year, Teeter extends to the owner a repair and replacement warranty against manufacturing defects in materials, workmanship, fabrics and padding. Teeter will repair or replace any such defect and will pay the costs of all parts, labor and transportation. If a repair or replacement is not commercially practical or cannot timely be made, then Teeter will, at the original Purchaser's option, replace with a comparable product or refund the purchase price.

Handling and transportation costs related to product warranty service only are covered by this warranty. This warranty does not cover damage resulting from improper handling, assembly, or installation, repairs made by others, accident, misuse, or abuse. Under no circumstances shall Teeter, or any other party involved in the sale of this product, have any liability for incidental or consequential damage arising from breach of an express or implied warranty on any Teeter product.

EXCEPT AS SET FORTH ABOVE, NO WARRANTY IS GIVEN WITH RESPECT TO ANY TEETER PRODUCT, AND ALL EXPRESS WARRANTIES ARE DISCLAIMED. To the extent this warranty is found not to be enforceable, it shall be deemed revised to the extent necessary to make it enforceable. This warranty and any controversy or claim arising out of this warranty or its interpretation shall be governed by the laws of the State of Washington, USA. Any controversy or claim arising out of or relating to this warranty, its interpretation, or any alleged breach thereof, which cannot be amicably settled between Teeter and the owner within sixty (60) days of written notice by the aggrieved party to the other, shall be finally settled by arbitration submitted to three (3) arbitrators selected from the panels of the arbitrators of the American Arbitration Association located closest to Teeter's principal place of business.

Some states do not allow the exclusion of incidental or consequential damage from a warranty, so the above limitation or exclusion may not apply to you. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state. This warranty is completely transferable to any and all future owners of this product, provided no alterations have been made to the product.

HOW TO SUBMIT YOUR REGISTRATION:

Step 1

Fill out this information for your own records.

_____ Date of Purchase

_____ Product & Model

Step 2

Go online to teeter.com to register your warranty.

_____ Dealer Name

_____ Serial No.

If you are unable to go online, you can request a warranty card to be mailed to you by calling Customer Service at 800-847-0143. Please DO NOT mail this to Teeter.

PLEASE RETAIN THIS FOR YOUR RECORDS

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International: Teeter International, Ltd.