





EZ-Angle™ Tether

ASSEMBLY & USE INSTRUCTIONS

HOW TO ASSEMBLE

- IF REPLACING A TETHER: Remove the pre-assembled Clip from the old Tether. Set aside for use with your new Tether.
- IF RE-ASSEMBLING AN EXISTING TETHER: Remove the pre-assembled Clip from the Strap Loop and set aside.
- Hold the Tether with Angle Markers facing up. Feed the Strap Loop through the FRONT of the A-frame U-Bar so that half is on one side and half on the other.
- Feed the Strap Loop through the bottom opening of the Sliding Buckle and then again through the top opening.
- Continue pulling on the Strap Loop until fitted snugly around the A-Frame U-Bar.
- Attach the Clip to the Strap Loop and secure through the Tether U-Bar on the underside of the Table Bed.

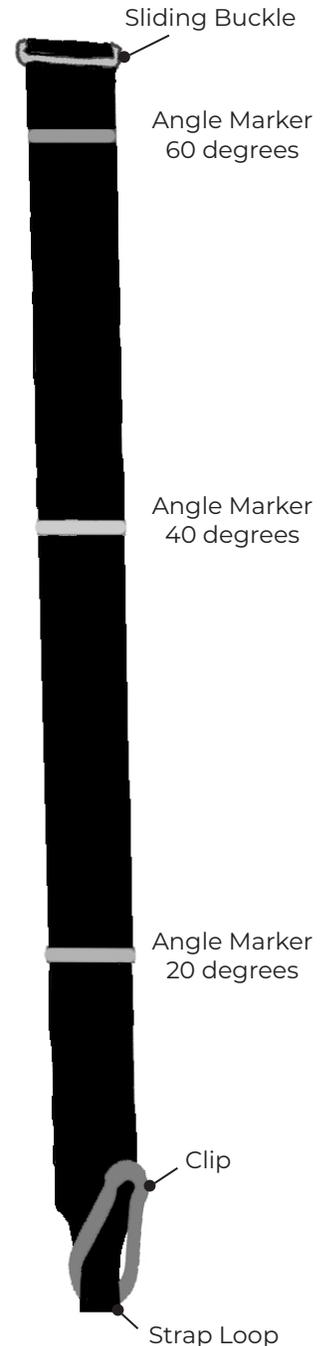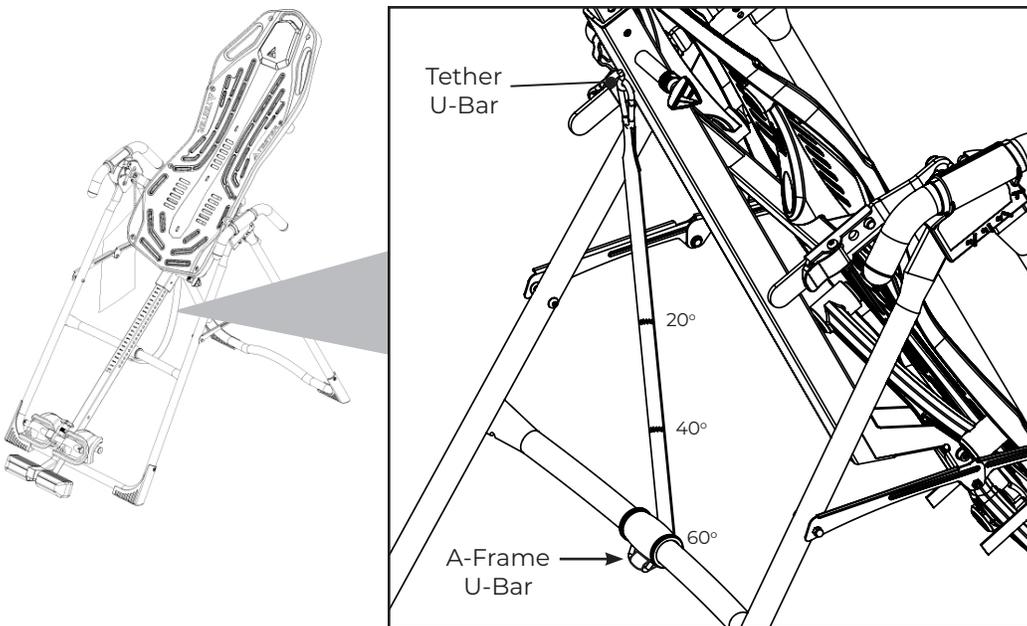

HOW TO USE

Adjust the length of the Tether by sliding the Buckle to your desired Angle Marker:

- 20 degrees - angle marker closest to Clip
- 40 degrees - middle angle marker
- 60 degrees - angle marker furthest from the Clip
- 90 degrees (full inversion) - detach the Clip from the Tether U-Bar

If you have any trouble assembling the equipment, or questions about its use, please contact Teeter customer service.

Phone: 800-847-0143 | Email: info@teeter.com