## Contour Series Inversion Tables

For gravity-assisted stretching and decompression

FASTER, BETTER
RESULTS WITH
TEETER SUPPORT
& COACHING



### **ASSEMBLE FASTER**

Free BILT® app puts easy 3-D instructions at your fingertips. Search 'Teeter' in BILT® to get started.



Find **BILT**® in your App Store



### EASY ACCESS TO VIDEO

Access videos online for easy assembly guidance, how-to's and classes.



Scan or

Go to teeter.com/videos



#### FREE EXERCISE GUIDE

Register your warranty and gain access to our 30-Day Back Pain Relief Guide.



Scan or

Go to teeter.com/warranty



### **REACH YOUR GOALS**

Use the free Teeter Move™ app to access guided sessions, product support, and more!





Find **Teeter Move**™ in your App Store



### WELCOME TO THE TEETER FAMILY

Congratulations on your purchase of a Teeter Contour™ Inversion Table! For the best experience, it is critical that you follow the assembly instructions, and read and fully understand the Owner's Manual attached to the equipment before inverting. Teeter Decompression Devices are multiple user, reusable medical devices for home use, intended to provide traction to the spine while stretching the para-spinal muscle and soft tissues. The devices provide non-powered traction and are meant for use by adults.

General Sector of 1 created Teeter so people could live healthier and more a<u>ctive liv</u>es. <sup>99</sup> -Roger Teeter

Teeter inversion tables are FDA-Registered as 510(k) medical devices. Indicated for:



- Back pain
- Muscle tension
- Degenerative disc disease
- Spinal degenerative joint disease
   Muscle spasm
- Spinal stenosis

- Herniated disc
- Spinal curvature due to tight muscles
- Sciatica
- Facet syndrome

### We've Got Your Back!



If you have any questions concerning assembly or if any parts are missing, DO NOT RETURN THE ITEM TO THE STORE OR CONTACT THE RETAILER. Our dedicated product service experts can help! Contact Teeter Customer Service at 800-847-0143, or via online forms or Live Chat at teeter.com.



Trust Teeter for unmatched quality and performance, backed by our industry-leading warranty coverage. To register your product warranty, go to teeter.com/Support/Warranty-Registration

### **Assembly Instructions**

Important Safety Instructions

important safety instructions
Items for Assembly
Understanding Your Inversion Table
Safety Warning Labels & Product Specifications 4
Before Beginning Assembly5
Assembly Steps
Before Inverting
Warranty Terms & Registration

### Owner's Manual (attached to the equipment)

Important Safety Instructions
User Settings
Prepare to Invert3 – «
Inverting4-
Storage & Maintenance
Get the Most Out of Your Teeter

**BEFORE YOU BEGIN:** Review all steps before beginning assembly and read all precautions before using the inversion table. Carefully adhere to the Assembly Instructions and Owner's Manual to help ensure safety and product integrity.

### IMPORTANT SAFETY INSTRUCTIONS READ ALL INSTRUCTIONS BEFORE USING THE INVERSION TABLE

### **AWARNING**

#### FAILURE TO FOLLOW INSTRUCTIONS AND WARNINGS COULD RESULT IN SERIOUS INJURY OR DEATH.

To reduce the risk of injury:

- Read and understand all the instructions, review all other accompanying documents, and inspect the equipment before using the inversion table. It is your responsibility to familiarize yourself with the proper use of this equipment and the inherent risks of inversion if these instructions are not followed, such as falling on your head or neck, pinching, entrapment, equipment failure, or aggravating a pre-existing medical condition. It is the responsibility of the owner to ensure that all users of the product are fully informed about the proper use of the equipment and all safety precautions.
- **DO NOT** use until approved by a licensed physician. Inversion is contraindicated in any medical or health condition that may be made more severe by an elevation of blood pressure, intracranial pressure or mechanical stress of the inverted position, or that may impact your ability to operate the equipment. This may include injury or illness, but also the side effects of any drug or supplement (prescribed or over-the-counter). Specific conditions may include, but not be limited to:
  - Any condition, neurological or otherwise, which results in unexplained tingling, weakness or neuropathy, seizure, sleep disorder, lightheadedness, dizziness, disorientation, or fatigue, or impacts strength, mobility, alertness, or cognitive ability;
  - · Any brain condition, such as trauma, history of intracranial bleed, history or risk of TIA or stroke, or severe headaches;
  - Any condition of the heart or circulatory system, such as high blood pressure, hypertension, increased risk of stroke, or use of anticoagulants (including high doses of aspirin);
  - Any bone, skeletal or spinal cord condition or injury, such as significant spinal curvature, acutely swollen joints, osteoporosis, fractures, dislocations, medullary pins or surgically implanted orthopedic supports;
  - Any eye, ear, nasal or balance condition, such as trauma, history of retinal detachment, glaucoma, optic hypertension, chronic sinusitis, middle or inner ear disease, motion sickness, or vertigo;
  - · Any digestive or internal condition, such as severe acid reflux, hiatal or other hernia, gallbladder or kidney disease;
  - Any condition for which exercise is specifically directed, limited or prohibited by a physician, such as pregnancy, obesity, or recent surgery.
- **ALWAYS** be certain the Ankle Lock System is properly adjusted and fully engaged, and that your ankles are secure before using the equipment. HEAR, FEEL, SEE and TEST that the Ankle Lock System is snug, close-fitting and secure EVERY TIME you use the equipment.
- · **ALWAYS** wear securely tied lace-up shoes with a flat sole, such as a normal tennis-style shoe.
- **DO NOT** wear any footwear that could interfere with securing the Ankle Lock System, such as shoes with thick soles, boots, high-tops or any shoe that extends above the anklebone.
- **DO NOT** use the inversion table until it is adjusted properly for your height and body weight. Improper settings can cause rapid inversion or make returning upright difficult. New users, and users who are physically or mentally compromised, will require the assistance of a spotter. Make sure the equipment is set to your unique user settings prior to each use.
- **DO NOT** sit up or raise head to return upright. Instead, bend knees and slide your body to the foot-end of the inversion table to change weight distribution. If locked out in full inversion, follow the instructions for releasing from the locked position before returning upright.
- **DO NOT** continue using the equipment if you feel pain or become light-headed or dizzy while inverting. Immediately return to the upright position for recovery and eventual dismount.
- **DO NOT** use if you are over 6 ft 6 in (198 cm) or over 300 lbs. (136 kg). Structural failure could occur or head/neck may impact the floor during inversion.
- **DO NOT** allow children to use this machine. Keep children, bystanders, and pets away from machine while in use. The inversion table is not intended for use by persons with reduced physical, sensory or mental capabilities, unless they are given supervision and instruction concerning use of the machine by a person responsible for their safety.
- DO NOT store the inversion table upright if children are present. Fold and lay the table on the floor. DO NOT store outdoors.
- **DO NOT** use aggressive movements, or use weights, elastic bands, any other exercise or stretching device or non-Teeter® attachments while on the inversion table. Use the inversion table only for its intended use as described in this manual.
- DO NOT drop or insert any object into any opening. Keep body parts, hair, loose clothing and jewelry clear of all moving parts.
- $\cdot \, \textbf{DO NOT} \, \text{use in any commercial, rental or institutional setting. This product is intended for indoor, home-use only.} \\$
- DO NOT operate equipment while under the influence of drugs, alcohol, or medication that may cause drowsiness or disorientation.
- · ALWAYS inspect the equipment prior to use. Make sure all fasteners are secure.
- $\cdot \textbf{ALWAYS} \ \text{replace defective components immediately and/or keep the equipment out of use until repair.}$
- · ALWAYS position equipment on a level surface and away from water or ledges that could lead to accidental immersion or falls.
- Refer to additional warning notices posted on the equipment. If a product label or Owner's Manual should become lost, damaged or illegible, contact Customer Service for replacement.

### **Items for Assembly**

Items not shown to scale. Hardware drawings located on the insert inside each Hardware Kit.

#### ITEM NO. ITEM NAME

### FlexTechnology™ Table Bed Assembly

NX1300 Table Bed

HK1010 Table Bed Assembly Hardware Kit

#### A-Frame Base Assembly

NX1100 A-Frame

F51008 EZ-Angle Tether (Pre-assembled to A-Frame)

#### **Handle Assembly**

Contour Stretch™ Handles (2) NX1500 HK1012 Handle Assembly Hardware Kit

#### Crossbrace Assembly

NX1400 Crossbrace

HK1011 Crossbrace Assembly Hardware Kit

#### ITEM NO. ITEM NAME

#### **Main Shaft Assembly**

NX1630 with EZ-Reach™ Ankle Lock System (Contour L3) NX1620 with Deluxe EZ-Reach™ Ankle Lock System (Contour L5)

#### **Optional Accessories**

IA8105 Head Pillow

#### **Tools Provided for Assembly**

10/13mm Open-Ended Wrenches (2) F51088


5mm Allen Wrenches (2) IA1149 EP1128A 6mm Allen Wrench (1)

#### **Product Support**

CN1730 Owner's Manual (Contour L3) -Pre-assembled CN1530 Owner's Manual (Contour L5) \_ to A-frame


#### FlexTechnology™ Table Bed Assembly

Use with Table Bed Assembly Hardware Kit (HK1010)

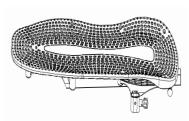

NX1300

#### A-Frame Base Assembly with pre-assembled EZ-Angle Tether

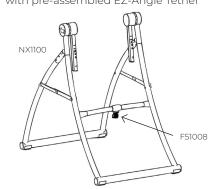

Handle Assembly Use with Handle Assembly Hardware Kit (HK1012)



NX1500

### Crossbrace Assembly

Use with Crossbrace Assembly Hardware Kit (HK1011)



NX1400

### Main Shaft

with EZ-Reach™ Ankle Lock System (Contour L3 Model Only)



### Main Shaft

with Deluxe EZ-Reach™ Ankle Lock System (Contour L5 Model Only)



### Tools Provided for Assembly

### Optional Head Pillow



IA8105







### Open-Ended Wrenches



#### F51088

6mm Allen Wrench

FP1128A

### 5mm Allen Wrenches

### IA1149


### Owner's Manual

CN1730 (Contour L3) CN1530 (Contour L5)

**Product Support** 

# Understanding Your Inversion Table Before reading further, study the drawing below to familiarize yourself with the

important components of your Teeter Inversion Table.

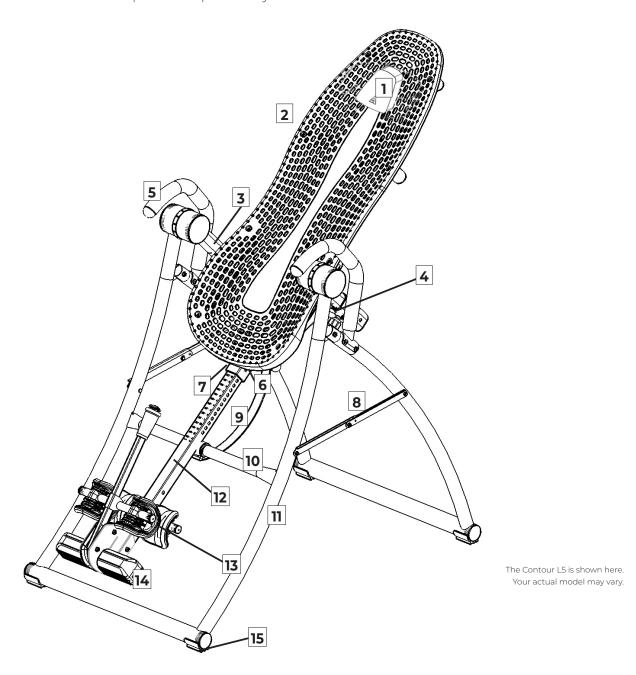

### Identifying Parts and Components

1	Head Pillow
2	FlexTechnology™ Table Bed
3	Rotation Adjustment Arm
4	Crossbrace
5	Contour Stretch™ Handles
6	Height-Selector Locking Pin

7	De-Rattler Knob
8	Spreader Arms
9	EZ-Angle™ Tether
10	Crossbar
11	A-Frame
12	Main Shaft


13	Ankle Lock System
14	Ankle Comfort Dial™
15	Stability Feet

### **Safety Warning Labels & Product Specifications**

Important: Please review all labels and supporting materials before using your inversion table.

This drawing indicates the locations of the warning labels found on your product. If a label is missing, illegible or is removed, contact Teeter Customer Service to request a complimentary replacement label.

Note: Image and labels below not shown at actual size.

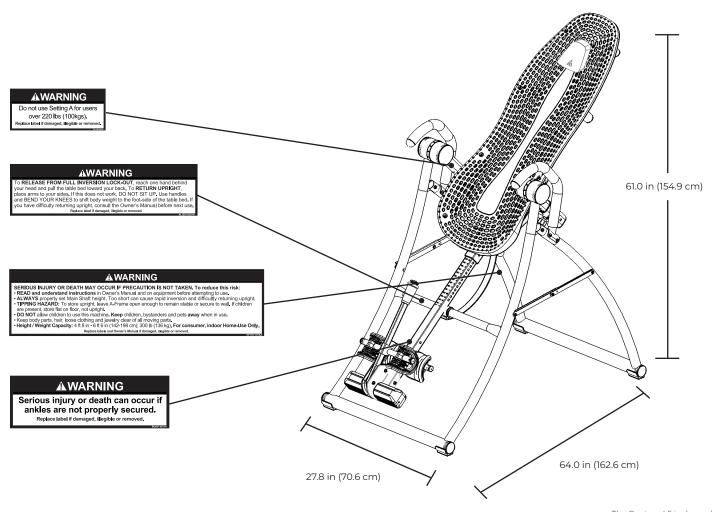

The Contour L5 is shown here. Your actual model may vary.

Assembled Non-Use Dimensions:  $64.0 \text{ (L)} \times 27.8 \text{ (W)} \times 61.0 \text{ in (H)} (162.6 \times 70.6 \times 154.9 \text{ cm)}$  Maximum In-Use Dimensions:  $83.0 \text{ (L)} \times 27.8 \text{ (W)} \times 88.0 \text{ in (H)} (210.8 \times 70.6 \times 223.5 \text{ cm)}$  Storage Dimensions:  $25.0 \text{ (L)} \times 27.8 \text{ (W)} \times 68.0 \text{ in (H)} (63.5 \times 70.6 \times 172.7 \text{ cm)}$ 

Weight (approx.): L3: 82.0 lbs (37.2 kg)

L5: 80.5 lbs (36.5 kg)

### **Before Beginning Assembly**

### **Unpack and Prepare Your Workspace**

- · If possible, assemble the equipment at or near the space in which you intend to use it to avoid moving it later.
- ·Unpack all parts and support materials. Set aside packing materials and clear your work area.
- $\cdot \text{Locate the Hardware Kits packaged with the manuals. They are labeled to correspond with the assembly process.}\\$

## Making Assembly Even Easier with **BILT**.



For step-by-step, 3D interactive instructions, download BILT (a FREE mobile app) to your smartphone to follow along. Simply download the BILT app by scanning the QR code below and then search for your model (L3 or L5) within the BILT app to get started!









## STEP ①


### **Attach Support Beam to Table Bed**

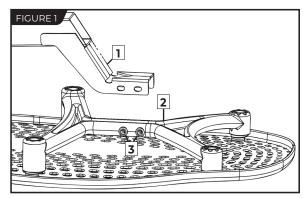

1 - Support Beam 2 - Upper Support Arms 3 - Rubber Spacers

- · Locate the Table Bed Assembly Hardware Kit (HK1010)
- · Place the FlexTechnology™ Table Bed face down on the floor.
- Push down on the Support Beam so that the two holes align evenly with the holes at the base of the Upper Support Arms (Figure 1).
- · Insert the two bolts into the open holes (Figure 2).
- Hand tighten each bolt with a washer and nut (Figure 3). Use the 6 mm Allen Wrench to steady each bolt and tighten the nuts using the 10/13 mm Open-Ended Wrench provided.

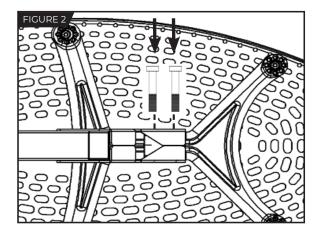

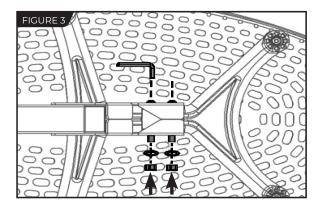


## STEP ②

### **Assemble Table Bed to A-Frame**


- Position the A-Frame on the floor with the crossbar pointing down, so that the frame is resting on the Rotation Adjustment Arms (Figure 4).
- With the Table Bed face up and the top of the bed pointing to the A-Frame feet, slide it on the floor between the legs of the A-Frame.
- · Loosen the "de-rattler" function of the Pinlock Knobs on each Support Housing by turning each counterclockwise (Figure 5).
- · Align and insert the Rotation Adjustment Arms into the Support Housings (Figure 6).
- Pull out both spring-loaded Pinlock Knobs at the same time then lift up to align the Support Housings with one of the marked settings on the Rotation Adjustment Arms (Figure 7).

NOTE: Refer to the Owner's Manual for an explanation of the hole settings. If you are unsure, use Setting C to start.


- Release the knobs so that they engage the same hole setting on each side (Figure 7). Should the knobs remain locked in the "out" position, simply rotate slightly clockwise to release (Figure 8).
- $\cdot$  Leave the inversion table in this position for the next step.

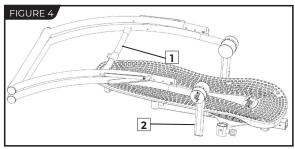


1 - Crossbar 2 - Rotation Adjustment Arm

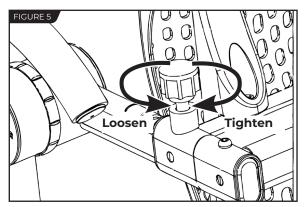

**Derattler Function**: Counterclockwise to loosen, clockwise to tighten

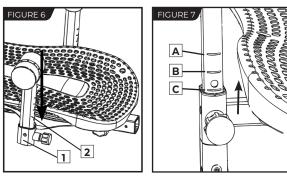


- $\hbox{\bf 1} \hbox{-} {\sf Support Housing}$
- **2** Rotation Adjustment Arms

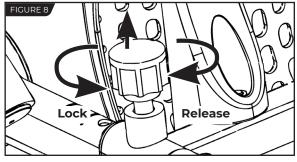


Pinlock Function - Pull out, turn counterclockwise to lock, clockwise to release

## STEP 3

### **Attach Handles to A-Frame**

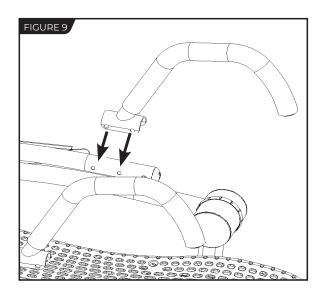

- · Locate the Contour Stretch™ Assembly Hardware Kit (HK1012).
- · With the open end of each handle pointing away from the A-Frame feet, place each Handle fitting over the A-Frame legs, aligning with the pre-drilled holes (Figure 9).
- · Insert the bolts from the inside and secure with the nuts on the outside, tightening with the Allen Wrenches provided (Figure 10).

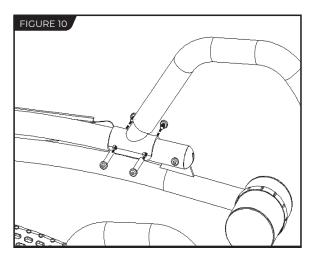





### **Attach Crossbrace to Rotation Adjustment Arms**

- · Locate the Crossbrace Hardware Kit (HK1011)
- $\cdot$  To stand the inversion table upright, first move the A-Frame onto its side (Figure 11), and then carefully lift it up onto its feet (Figure 12).
- · Use the handles to open the A-Frame wide enough so that it is stable and can stand upright on its own (Figure 13).
- · Align the holes in the Crossbrace with the last hole on the Rotation Adjustment Arms (Figure 14).
- · Insert the Chicago Posts and secure with the Chicago Screws, tightening with the Allen Wrenches provided (Figure 15).


**NOTE:** Due to the tapered style of the Chicago Post, it may not sit flush against the Crossbrace.


• Turn the Pinlock Knobs clockwise to re-engage the "de-rattler" function (Figure 16).

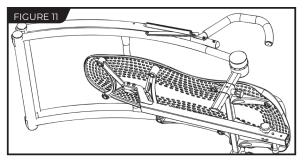


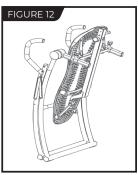



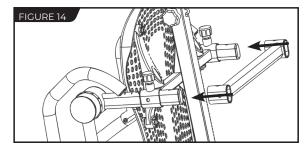


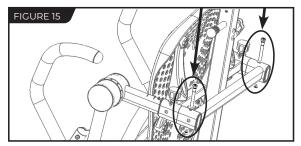


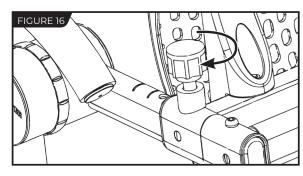


## STEP (5)

### **Assemble Main Shaft to Table Bed**

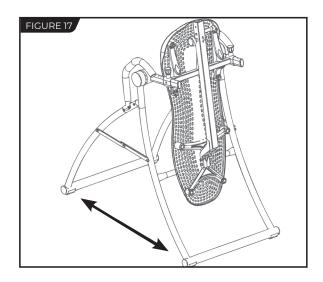

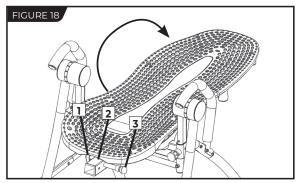

- 1 De-Rattler Knob 2 Main Shaft Housing
- 3 Height-Selector Locking Pin

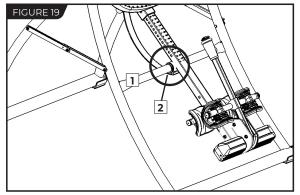

1 - Crossbar 2 - Proper Resting Position

- Ensure that the A-Frame is fully open and locked at the Spreader Arms (Figure 17).
- · Rotate the Table Bed into its use position (Figure 18).
- · Loosen the Main Shaft De-Rattler Knob at the base of the Main Shaft Housing.

**NOTE:** The De-Rattler Knob is an optional feature that prevents a slight shift when you invert. Product security is not compromised if you choose not to tighten the knob.

- · With the Height Adjustment Settings on the Main Shaft facing up, slide the end of the Main Shaft into the Main Shaft Housing at the base of the Table Bed.
- Pull out the Height-Selector Locking Pin to allow the Main Shaft to slide in further and release in the desired height setting. Refer to the Owner's Manual for more information on selecting your height setting.
- The Main Shaft MUST REST against the Crossbar of the A-Frame (Figure 19).

**NOTE:** The Crossbar prevents the Table Bed from rotating forward when the user steps on the Ankle Comfort Dial. If the Main Shaft does not rest on the Crossbar as shown in Figure 19, then the Table Bed has been assembled backwards onto the A-Frame. This must be corrected before use.

## STEP 6

## **Attach Angle Tether & Head Pillow**


### **Attach Angle Tether**

- The tether will come pre-assembled to the A-Frame.
- Unfold the adjustable tether and clip it to the U-Bar on the underside of the Table Bed (Figure 20).
- Slide the buckle to lengthen or shorten the strap depending on your desired maximum angle of inversion.




#### EZ-Angle Tether Accessory


Some models may come with the EZ-Angle Tether, embroidered with color-coded angle markers at 20° (GREEN), 40° (ORANGE), or 60° (RED). Simply slide the buckle so its center aligns with your desired color setting.

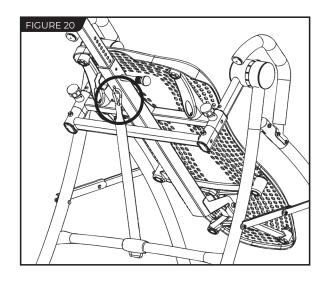


### **Attach Head Pillow**

Attach the Head Pillow by securing the Velcro Straps through the elongated slots in the Table Bed (Figure 21), which allow the pillow to shift with the user when in use. You may also customize the position depending on your preference.

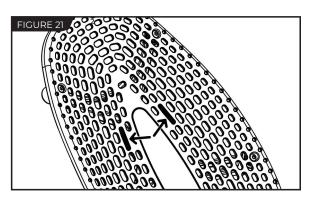

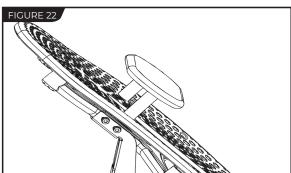

### **Before Inverting**

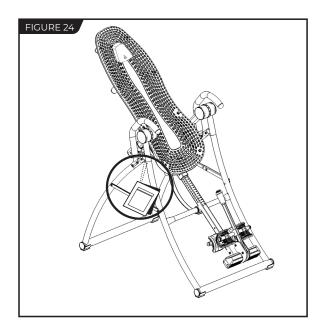

### **Ensure Owner's Manual is Attached**

The Owner's Manual contains important information on how to use your Teeter Inversion Table, including how to personalize the user settings, properly secure and release the Ankle Lock System, and test and adjust the rotation control.

- · If not already attached, thread the provided metal chain through the pre-punched hole in the upper corner of the Owner's Manual.
- Secure the chain to the A-Frame through the designated hole in the Hinge Plate (Figure 24). Allow the Owner's Manual to hang freely on the outside of the A-Frame Spreader Arms so it doesn't interfere with the rotation of the Table Bed.

IMPORTANT: Once attached to the A-Frame, DO NOT remove the Owner's Manual. It should remain permanently attached to your inversion table to serve as a reference for all users in regards to proper adjustment and use of the equipment.

### **AWARNING**

Read the Owner's Manual thoroughly before using your Teeter Inversion Table. Improper settings could result in serious injury or death!

### **Watch the Getting Started Videos**

The Getting Started Video Portal is a helpful supplement to the Owner's Manual, with easy-to-follow instructions on user settings, how to invert, storage and maintenance, and even stretching and exercises you can do with your Teeter. Access via teeter.com/videos and search your Teeter Model now!

The Teeter warranty set forth below and on Teeter's website applies to US and Canadian customers only. For international customers, please consult your local distributor for warranty information which will vary depending on country.



During the period starting with the day of retail purchase and continuing for five (5) years, Teeter extends to the owner a repair and replacement warranty against manufacturing defects in materials, workmanship, fabrics and padding. Teeter will repair or replace any such defect and will pay the costs of all parts, labor and transportation. If a repair or replacement is not commercially practical or cannot timely be made, then Teeter will, at the original Purchaser's option, replace with a comparable product or refund the purchase price.

Handling and transportation costs related to product warranty service only are covered by this warranty. This warranty does not cover damage resulting from improper handling, assembly, or installation, repairs made by others, accident, misuse, or abuse. Under no circumstances shall Teeter, or any other party involved in the sale of this product, have any liability for incidental or consequential damage arising from breach of an express or implied warranty on any Teeter product.

EXCEPT AS SET FORTH ABOVE, NO WARRANTY IS GIVEN WITH RESPECT TO ANY TEETER PRODUCT, AND ALL EXPRESS

**WARRANTIES ARE DISCLAIMED.** This warranty shall be governed by the laws of the State of Washington, USA. To the extent this warranty is found not to be enforceable, it shall be deemed revised to the extent necessary to make it enforceable. This warranty and any controversy or claim arising out of this warranty or its interpretation shall be governed by the laws of the State of Washington, USA. Any controversy or claim arising out of or relating to this warranty, its interpretation, or any alleged breach thereof, which cannot be amicably settled between Teeter and the owner within sixty (60) days of written notice by the aggrieved party to the other, shall be finally settled by arbitration submitted to three (3) arbitrators selected from the panels of the arbitrators of the American Arbitration Association located closest to Teeter's principal place of business.

Some states do not allow the exclusion of incidental or consequential damage from a warranty, so the above limitation or exclusion may not apply to you. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

#### **HOW TO SUBMIT YOUR REGISTRATION:**

#### Step 1

Fill out this information for your own records.

### Step 2

Go online to teeter.com to register your warranty.

Date of Purchase	
Product & Model	
Dealer Name	
erial No.	П

If you are unable to go online, you can request a warranty card to be mailed to you by calling Customer Service at 800-847-0143. Please DO NOT mail this to Teeter.





# **ATEETER**

If you have any trouble assembling the equipment, or questions about its use, please contact customer service.

USA: 800-847-0143 or info@teeter.com International: info@teeterintl.com

Check out the selection of products and accessories available at teeter.com!

USA: Teeter 9902 162<sup>nd</sup> St. Ct. E. Puyallup, WA 98375 Toll Free: 800-847-0143 Fax: 800-847-0188 teeter.com | info@teeter.com



Any modification to this device will void the UL Listing.

International: Teeter International, Ltd. Cor-Ray House 758 Great Cambridge Rd Enfield Middlesex ENI 3GN United Kingdom

teeterintl.com | info@teeterintl.com



Medical Device Safety Service GmbH Schiffgraben 41 30175 Hannover Germany Tel. +49 511 62628630



Teeter Decompression Devices are multiple user, reusable devices for home use, intended to provide traction to the spine while stretching the para-spinal muscles and soft tissues.

The devices provide non-powered traction and are meant for use by adults.

Use of FDA-Cleared Teeter Decompression Devices is indicated for the following conditions: back pain, muscle tension, degenerative disc disease, spinal degenerative joint disease, spinal stenosis, herniated disc, spinal curvature due to tight muscles, sciatica, muscle spasm, and facet syndrome.