

# **ATEETER**

# FS-1™ Inversion Table



IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY




To download and print the Teeter FS-1™ instructions in French, visit the Product Manuals page at teeter.com.



\* Specifications may vary from this image and are subject to change without notice.





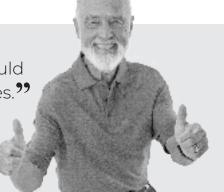

Congratulations on your purchase of a Teeter FS-1<sup>TM</sup> Inversion Table! For the best experience, it is critical that you follow the assembly instructions, read and fully understand the Owner's Manual attached to the equipment, and review the Getting Started Video Portal prior to using your new Teeter Inversion Table.

Teeter Decompression Devices are 510(k) registered with the FDA and are multiple user, reusable medical devices for home use, intended to provide traction to the spine while stretching the para-spinal muscle and soft tissues. The devices provide non-powered traction and are meant for use by adults.

Teeter Decompression Devices are indicated for the following conditions: back pain, muscle tension, degenerative disc disease, spinal degenerative joint disease, spinal stenosis, herniated disc, spinal curvature due to tight muscles, sciatica, muscle spasm, and facet syndrome.



For step-by-step, 3D interactive instructions, download BILT (a FREE mobile app) to your smartphone to follow along. See Page 5 for instructions on how to download BILT.



To register your product warranty, go to **teeter.com/Support/Warranty-Registration** 



If you have any questions concerning assembly or if any parts are missing, DO NOT RETURN THE ITEM TO THE STORE OR CONTACT THE RETAILER. Our dedicated product service experts can help! Contact Teeter Customer Service at **800.847.0143**, or via online forms or Live Chat at **teeter.com**.



Get more from your inversion sessions with **TeeterLink®**, the FREE app that lets you track usage and pain levels, get reminders and customized guidance, access support, and more! Scan the code or search TeeterLink® in your app store to get started!



#### **Assembly Instructions**

Important Safety Instructions
Parts for Assembly
Understanding Your Inversion Table 3
Safety Warning Labels & Product Specifications 4
Before Beginning Assembly
Assembly Steps6 – 11
Misassembly Check
Before Inverting
Warranty Terms & Registration

### Owner's Manual (attached to the equipment)

Important Safety Instructions	
User Settings	2
Prepare to Invert3 -	. ∠
Inverting4-	
Storage & Maintenance	_
Get the Most out of Your Teeter	6

**BEFORE YOU BEGIN:** Review all steps before beginning assembly and read all precautions before using the inversion table. Carefully adhere to the Assembly Instructions and Owner's Manual to help ensure safety and product integrity.

# IMPORTANT SAFETY INSTRUCTIONS READ ALL INSTRUCTIONS BEFORE USING THE INVERSION TABLE

### **AWARNING**

#### FAILURE TO FOLLOW INSTRUCTIONS AND WARNINGS COULD RESULT IN SERIOUS INJURY OR DEATH.

To reduce the risk of injury:

- Read and understand all the instructions, review all other accompanying documents, and inspect the equipment before using the inversion table. It is your responsibility to familiarize yourself with the proper use of this equipment and the inherent risks of inversion if these instructions are not followed, such as falling on your head or neck, pinching, entrapment, equipment failure, or aggravating a pre-existing medical condition. It is the responsibility of the owner to ensure that all users of the product are fully informed about the proper use of the equipment and all safety precautions.
- **DO NOT** use until approved by a licensed physician. Inversion is contraindicated in any medical or health condition that may be made more severe by an elevation of blood pressure, intracranial pressure or mechanical stress of the inverted position, or that may impact your ability to operate the equipment. This may include injury or illness, but also the side effects of any drug or supplement (prescribed or over-the-counter). Specific conditions may include, but not be limited to:
  - Any condition, neurological or otherwise, which results in unexplained tingling, weakness or neuropathy, seizure, sleep disorder, lightheadedness, dizziness, disorientation, or fatigue, or impacts strength, mobility, alertness, or cognitive ability;
  - · Any brain condition, such as trauma, history of intracranial bleed, history or risk of TIA or stroke, or severe headaches;
  - Any condition of the heart or circulatory system, such as high blood pressure, hypertension, increased risk of stroke, or use of anticoagulants (including high doses of aspirin);
  - Any bone, skeletal or spinal cord condition or injury, such as significant spinal curvature, acutely swollen joints, osteoporosis, fractures, dislocations, medullary pins or surgically implanted orthopedic supports;
  - Any eye, ear, nasal or balance condition, such as trauma, history of retinal detachment, glaucoma, optic hypertension, chronic sinusitis, middle or inner ear disease, motion sickness, or vertigo;
  - · Any digestive or internal condition, such as severe acid reflux, hiatal or other hernia, gallbladder or kidney disease;
  - Any condition for which exercise is specifically directed, limited or prohibited by a physician, such as pregnancy, obesity, or recent surgery.
- **ALWAYS** be certain the Ankle Lock System is properly adjusted and fully engaged, and that your ankles are secure before using the equipment. HEAR, FEEL, SEE and TEST that the Ankle Lock System is snug, close-fitting and secure EVERY TIME you use the equipment.
- · ALWAYS wear securely tied lace-up shoes with a flat sole, such as a normal tennis-style shoe.
- **DO NOT** wear any footwear that could interfere with securing the Ankle Lock System, such as shoes with thick soles, boots, high-tops or any shoe that extends above the anklebone.
- **DO NOT** use the inversion table until it is adjusted properly for your height and body weight. Improper settings can cause rapid inversion or make returning upright difficult. New users, and users who are physically or mentally compromised, will require the assistance of a spotter. Make sure the equipment is set to your unique user settings prior to each use.
- **DO NOT** sit up or raise head to return upright. Instead, bend knees and slide your body to the foot-end of the inversion table to change weight distribution. If locked out in full inversion, follow the instructions for releasing from the locked position before returning upright.
- **DO NOT** continue using the equipment if you feel pain or become light-headed or dizzy while inverting. Immediately return to the upright position for recovery and eventual dismount.
- **DO NOT** use if you are over 198 cm / 6 ft 6 in / 6 po 6 pi or over 136 kg (300 lb). Structural failure could occur or head / neck may impact the floor during inversion.
- **DO NOT** allow children to use this machine. Keep children, bystanders, and pets away from machine while in use. The inversion table is not intended for use by persons with reduced physical, sensory or mental capabilities, unless they are given supervision and instruction concerning use of the machine by a person responsible for their safety.
- · DO NOT store the inversion table upright if children are present. Fold and lay the table on the floor. DO NOT store outdoors.
- **DO NOT** use aggressive movements, or use weights, elastic bands, any other exercise or stretching device or non-Teeter® attachments while on the inversion table. Use the inversion table only for its intended use as described in this manual.
- · DO NOT drop or insert any object into any opening. Keep body parts, hair, loose clothing and jewelry clear of all moving parts.
- · DO NOT use in any commercial, rental or institutional setting. This product is intended for indoor, home-use only.
- · DO NOT operate equipment while under the influence of drugs, alcohol, or medication that may cause drowsiness or disorientation.
- · ALWAYS inspect the equipment prior to use. Make sure all fasteners are secure.
- · ALWAYS replace defective components immediately and / or keep the equipment out of use until repair.
- · ALWAYS position equipment on a level surface and away from water or ledges that could lead to accidental immersion or falls.
- Refer to additional warning notices posted on the equipment. If a product label or Owner's Manual should become lost, damaged or illegible, contact Customer Service for replacement.

### **Parts for Assembly**

Parts are not shown to scale. Hardware drawings located on the insert inside each Hardware Kit.

ITEM NO. ITEM NAME

Stretch-and-Grip™ A-Frame Base Assembly

E61100 A-Frame

F51007 Angle Tether pre-assembled to A-Frame

**Handle Assembly** 

E61500 Stretch Assist™ Handles (2)

HK1008 Handle Assembly Hardware Kit

F50071 Allen Head Bolt (6)

**Roller Hinge Assembly** 

F51064B Roller Hinges (2)

FitSpine™ Table Bed Assembly

EX1300 FitSpine™ Table Bed

HK1010 FitSpine™ Table Bed Hardware Kit

EP1127 Bolt (2), H13009 Nut (2), EP1128 Washer (2)

ITEM NO. ITEM NAME

**Main Shaft Assembly** 

EX1630 with EZ-Reach™ Ankle Lock System

**Optional Accessories** 

EX1800 Head Pillow

**Tools Provided for Assembly** 


IA1149 5mm Allen Wrench (1) EP1128A 6mm Allen Wrench (1)

F51088 Open-Ended Wrench (1)

**Product Support** 

LI3900 Owner's Manual pre-assembled to A-Frame

Stretch-and-Grip™ A-Frame Base Assembly with pre-assembled Angle Tether

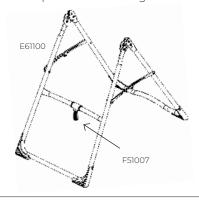

FitSpine™ Table Bed Assembly
Use with Table Bed Hardware Kit (HK1010)



Optional Head Pillow



5mm Allen Wrench

Handle Assembly
Use with Handle Assembly
Hardware ''it (HK1008)



Roller Hinge Assembly



F51064B

Main Shaft Assembly

EZ-Reach™ Ankle Lock System



**Product Support** 



Owner's Manual

Tools Provided for Asse

6mm Allen Wrench




Open-Finded-Wrench

F51088

# Understanding Your Inversion Table Before reading further, study the drawing below to familiarize yourself with the

important components of your Teeter Inversion Table.

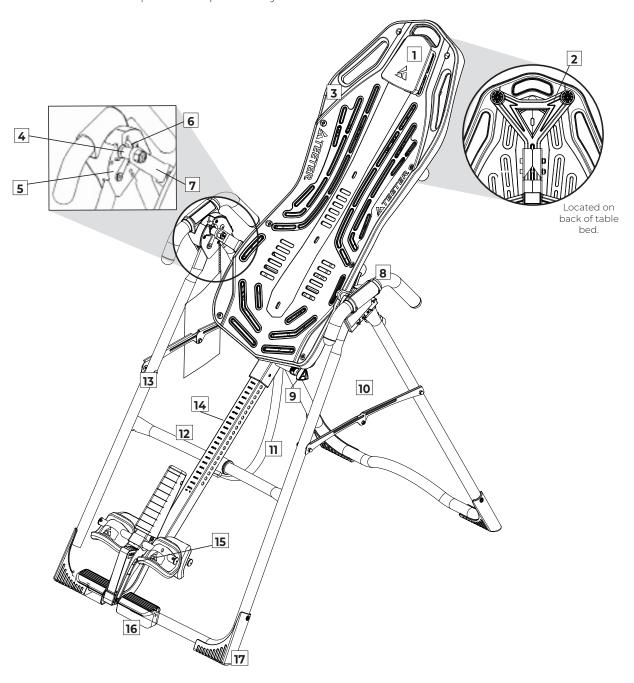

### Identifying Parts and Components

1	Head Pillow
2	Upper Support Arms
3	FitSpine™ Table Bed
4	Pivot Pins
5	Hinge Plates
6	Self-Locking Hooks


7	3-Hole Roller Hinges
8	Handles
9	Height-Selector Locking Pin
10	Spreader Arms
11	Angle Tether
12	Crossbar

13	A-Frame
14	Main Shaft
15	Ankle Lock System
16	Ankle Comfort Dial™
17	Non-Skid Stability Feet

# Safety Warning Labels & Product Specifications Important: Please review all labels and supporting materials before using your inversion table.

This drawing indicates the locations of the warning labels found on your product. If a label is missing, illegible or is removed, contact Teeter Customer Service to request a complimentary replacement label.

Note: Image and labels below not shown at actual size.

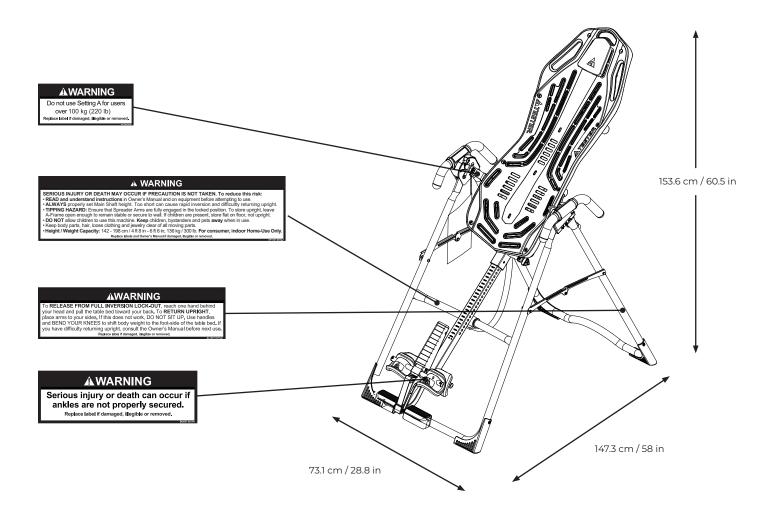

Assembled Non-Use Dimensions: 147.3 x 73.1 x 153.6 cm / 58 x 28.8 x 60.5 in Maximum In-Use Dimensions: Storage Dimensions:

205.7 x 73.1 x 219.7 cm / 81 x 28.8 x 86.5 in 50.8 x 73.1 x 167.6 cm / 20 x 28.8 x 66 in

Weight (approx.): 28.4 kg / 62.6 lb

### **Before Beginning Assembly**

### **Unpack and Prepare Your Workspace**

- · If possible, assemble the equipment at or near the space in which you intend to use it to avoid moving it later.
- · Unpack all parts and support materials. Set aside packing materials and clear your work area.
- · Locate the Hardware Kits packaged with the manuals. They are labeled to correspond with the assembly process.
- The Getting Started Video Portal at teeter.com/videos provides step-by-step instruction on how to assemble your product. You may find it helpful to follow along with the videos by watching them on either your phone, tablet, or computer. The videos include important use instructions such as:
  - · Assembly Follow step-by-step instruction on how to assemble your Teeter
  - · User Settings Personalize your inversion experience by adjusting four customizable settings.
  - · How to Invert Learn how to test your balance and rotation control, and how to properly invert and return upright.
  - · Dismounting Learn how to properly dismount your Teeter according to your specific model.
  - · Storage & Maintenance When not in use, store your Teeter easily and compactly.
  - · Accessories Learn more about the accessories available for your Teeter.



· Healthy Back and Body Routines\*

Get guided instruction on stretches and exercises you can do on your own to help strengthen your back and gain flexibility. Access this exclusive bonus content at **teeter.com/videos** with the password: *healthyback*.

\*Not available in French or Spanish.

# Making Assembly Even Easier with **BILT**.



For step-by-step, 3D interactive instructions, download BILT (a FREE mobile app) to your smartphone to follow along. Simply download the BILT app by scanning the QR code below and then search for **Teeter FS-1** within the BILT app to get started!









# STEP ①

### **Assemble A-Frame Base & Stretch Assist Handles**

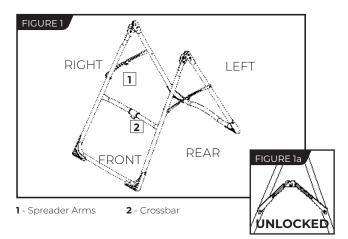




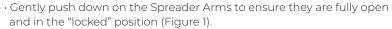

 Look for temporary circular assembly assistance labels on the A-Frame. RIGHT, LEFT, FRONT, and REAR indicate your position while using the equipment, not facing it. These labels can be removed easily upon completion of assembly.




 $\cdot$  Determine the left or right handles, marked with an embossed L / R on the inside of the black plastic part of each handle.

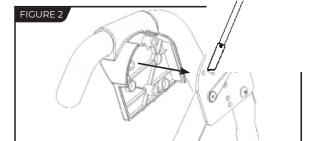

ign the black plastic part of the corresponding handle (left / right) rer the outside edge of the Hinge Plate on the A-Frame (Figure 2).


Sert and loosely hand-tighten three of the Allen Head Screws through the Hinge Plate into the handle (Figure 3).

· Repeat with other handle. Tighten all fasteners with the 5mm Allen Wrench provided, being careful not to over-tighten.

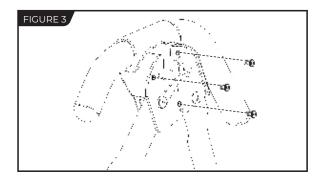


# STEP ②

### **Assemble Table Bed**

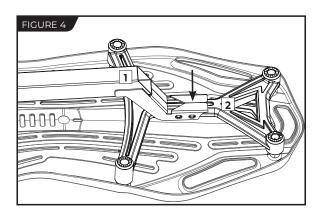

- 1 Support Beam
- 2 Upper Support Arms

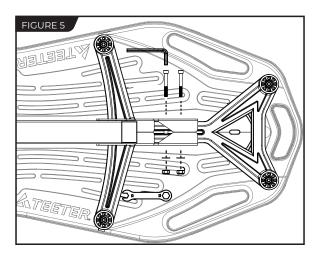

- · Locate the Table Bed Assembly Hardware Kit (HK1010).
- Place the Table Bed face down on the floor and push down on the Support Beam, so the two holes align evenly with the holes at the base of the Upper Support Arms (Figure 4). You may have to exert extra pressure to ensure that the Support Beam slides over the rubber spacers.
- · Insert the two Bolts into the open holes (Figure 5).
- · Thread a Washer and Nut onto each Bolt and hand-tighten.
- · Using the 6mm Allen Wrench to steady the bolts, tighten the Nuts onto the Bolts with the 10/13mm Open-Ended Wrench.

# STEP 3

# **Assemble Roller Hinges to Table Bed**

• Familiarize yourself with the 3-Hole Roller Hinge and Cam Lock terms (Figure 6).

## **AWARNING**

NEVER disassemble the Roller Hinge Pivot Pin.
ALWAYS insert the 3-Hole Roller Hinge (with the Pivot Pin on top and facing out) in the same direction as the arrow label located inside of the Cam Lock for proper assembly.


- · For ease of assembly, rest the Table Bed against the Crossbar (Figure 7) at the front of the A-Frame.
- · On one side of the Table Bed, lift and hold the Cam Lock up all the way to unlock (Figure 8).
- In your other hand, hold one Roller Hinge near the Pivot Pin. With the Pivot Pin facing out (away from the Table Bed), slide the bottom of the Roller Hinge between the Cam Lock and the Bracket in the same direction as the arrow label located inside of the Cam Lock.


TIP: Make sure that the Cam Lock is completely open when inserting the Roller Hinge, otherwise assembly will be more difficult.


• Engage one of the holes in the Roller Hinge over the Bracket Pin. Figure 10 shows the Roller Hinge installed correctly, with the Bracket Pin engaged in Setting C.


**NOTE**: Refer to the Owner's Manual for an explanation of the hole settings. If you are unsure, use Setting C to start.


- $\cdot$  Push down on the Cam Lock (Figure 9) to lock it and secure the Roller Hinge.
- Repeat on other side. Make sure the Roller Hinges are in the same hole setting on both sides.

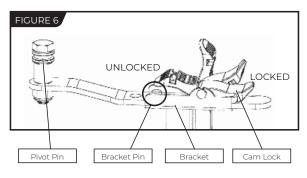

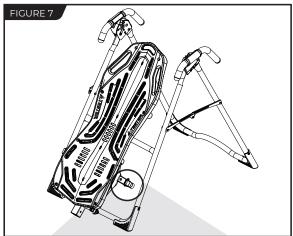

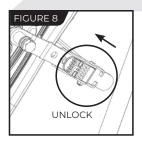


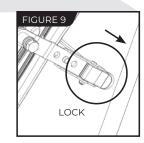


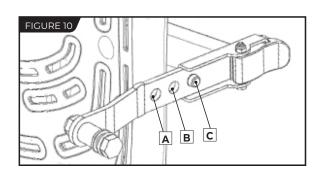



# STEP 4


## **Assemble Table Bed to A-Frame**

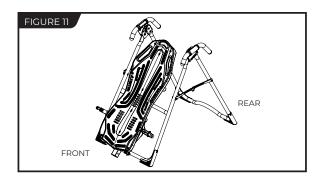


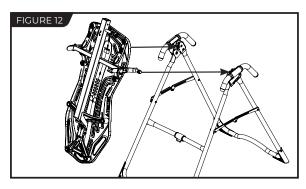

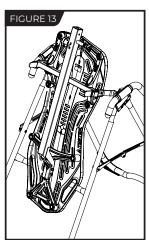

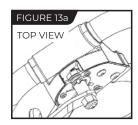

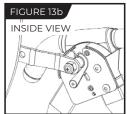

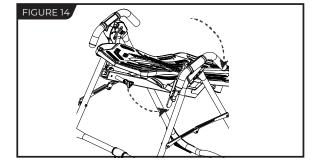

- · Face the front of the A-Frame where the Crossbar is located (Figure 11).
- Grasp both Roller Hinges, right above the Cam Lock, and lift the Table Bed. Allow the top of the Table Bed to rotate toward the floor, so that the back of the Table Bed is now facing you and the top of the Table Bed is in front of the Crossbar (Figure 12).
- · Lower each Roller Hinge Pivot Pin into the A-Frame hinge plates, one side at a time (Figure 13). The Self-Locking Hooks will open to allow the Pivot Pin into the Hinge Plate slot, then automatically snap closed over the Pivot Pin.

TIP: You may need to push outward on the Hinge Plate in order for the second Pivot Pin to lock in place.

• Make sure that each Pivot Pin is seated at the base of the slot in the Hinge Plates, and that the Self-Locking Hooks have closed over both Pivot Pins (Figures 13a & 13b).

### **AWARNING**

Failure of the Self-Locking Hooks to close over both Roller Hinge Pivot Pins is an indication of improper assembly and if not corrected could result in serious injury or death!

Rotate the Table Bed into the use position (Figure 14). Ensure that it rotates smoothly. See also Image A on Page 12 to ensure correct assembly.

# STEP (5)


### **Assemble Main Shaft to Table Bed**


- · Facing the front of the A-Frame, hold the Main Shaft in your left hand with the height markings facing up. Slide the end of the Main Shaft into the Main Shaft Housing (Figure 15), located at the base of the Table Bed.
- · With your right hand, pull out the Height-Selector Locking Pin (Figure 16) to allow the Main Shaft to slide in further and release in the desired height setting. Refer to the Owner's Manual for more information on selecting your height setting.
- The Main Shaft MUST REST against the Crossbar bumper on the A-Frame (Figure 17).


IMPORTANT: The Crossbar prevents the Table Bed from rotating forward when the user steps on the Ankle Comfort Dial. If the Main Shaft does not rest on the Crossbar bumper as shown in Figure 16, then the Table Bed has been assembled backwards onto the A-Frame.


This MUST BE CORRECTED before use. See also Image B on Page 12 to ensure correct assembly.

 $\cdot$  Test the inversion table by hand for smooth and steady rotation (Figure 18) and ensure that all fasteners are secure.

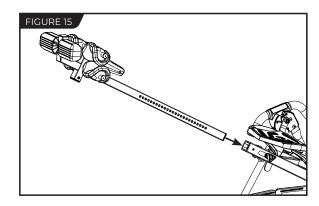

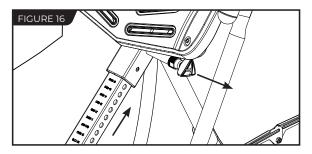


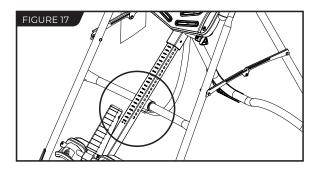

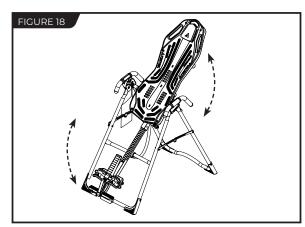

# STEP 6


# **Attach Angle Tether & Head Pillow**

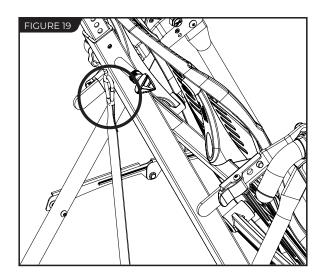

### **Attach Angle Tether**

- The tether will come pre-assembled to the A-Frame.
- Unfold the adjustable tether and clip it to the U-Bar on the underside of the Table Bed (Figure 19).
- · Slide the buckle to lengthen or shorten the strap depending on your desired maximum angle of inversion.



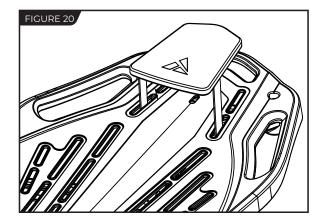

### **Attach Head Pillow**

Attach the Head Pillow by securing the hook and loop straps through the specified holes in the Table Bed (Figure 20), which allow the pillow to shift with the user when in use. You may also customize the position depending on your preference.

# **Misassembly Check**

# **AWARNING**

If your Teeter Inversion Table looks like either of these images, your inversion table has been misassembled and is unfit for use. Improper assembly could result in serious injury or death!

### **Image A**


### Go back to Step 3 for instruction.


Demonstrates that the Roller Hinges have been assembled upside down into the Table Bed and must be corrected.

### **Image B**


### Go back to Step 4 for instruction.


Demonstrates that the Table Bed has been assembled into the A-Frame backwards so the Main Shaft is not resting on the Crossbar and must be corrected.

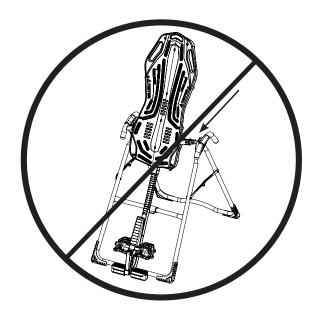

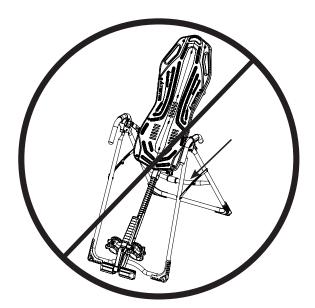

## **Before Inverting**

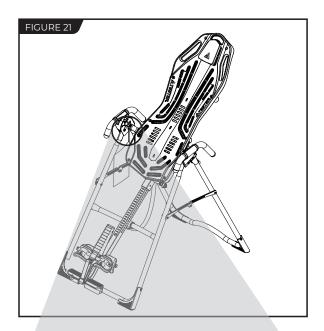

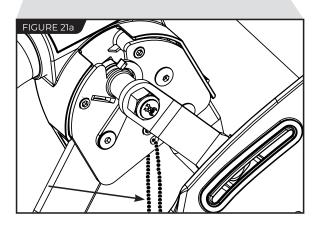

### **Ensure Owner's Manual is Attached**

The Owner's Manual contains important information on how to use your Teeter Inversion Table, including how to personalize the user settings, properly secure and release the Ankle Lock System, and test and adjust the rotation control.

- · If not already attached, thread the provided metal chain through the pre-punched hole in the upper corner of the Owner's Manual.
- Secure the chain to the A-Frame through the designated hole in the Hinge Plate (Figure 21 & 21a). Allow the Owner's Manual to hang freely on the outside of the A-Frame Spreader Arms so it doesn't interfere with the rotation of the Table Bed.

IMPORTANT: Once attached to the A-Frame, DO NOT remove the Owner's Manual. It should remain permanently attached to your inversion table to serve as a reference for all users in regards to proper adjustment and use of the equipment.

## **AWARNING**

Read the Owner's Manual thoroughly before using your Teeter Inversion Table. Improper settings could result in serious injury or death!

### **Watch the Getting Started Videos**

The Getting Started Video Portal is a helpful supplement to the Owner's Manual, with easy-to-follow instructions on user settings, how to invert, storage and maintenance, and even stretching and exercises you can do with your Teeter. Just go to teeter.com/videos to access them now!

### **Download TeeterLink®**

Get more from your inversion sessions by downloading the TeeterLink® app to your phone to track usage and pain levels, get reminders and customized guidance, access support, and more! Scan the code or search TeeterLink® in your app store to get started!





The Teeter warranty set forth below and on Teeter.com applies to the Inversion Table only. Accessories may come with their own terms. This warranty applies to US and Canadian customers only. For international customers, please consult your local distributor for warranty information which will vary depending on country.



During the period starting with the day of retail purchase and continuing for five (5) years, Teeter extends to the owner a repair and replacement warranty against manufacturing defects in materials, workmanship, fabrics and padding. Teeter will repair or replace any such defect and will pay the costs of all parts, labor and transportation. If a repair or replacement is not commercially practical or cannot timely be made, then Teeter will, at the original Purchaser's option, replace with a comparable product or refund the purchase price.

Handling and transportation costs related to product warranty service only are covered by this warranty. This warranty does not cover damage resulting from improper handling, assembly, or installation, repairs made by others, accident, misuse, or abuse. Under no circumstances shall Teeter, or any other party involved in the sale of this product, have any liability for incidental or consequential damage arising from breach of an express or implied warranty on any Teeter product.

#### EXCEPT AS SET FORTH ABOVE, NO WARRANTY IS GIVEN WITH RESPECT TO ANY TEETER PRODUCT, AND ALL

**EXPRESS WARRANTIES ARE DISCLAIMED.** This warranty shall be governed by the laws of the State of Washington, USA. To the extent this warranty is found not to be enforceable, it shall be deemed revised to the extent necessary to make it enforceable. Any controversy or claim arising out of or relating to this warranty, its interpretation, or any alleged breach thereof, which cannot be amicably settled between Teeter and the owner within sixty (60) days of written notice by the aggrieved party to the other, shall be finally settled by arbitration submitted to three (3) arbitrators selected from the panels of the arbitrators of the American Arbitration Association located closest to Teeter's principal place of business.

Some states do not allow the exclusion of incidental or consequential damage from a warranty, so the above limitation or exclusion may not apply to you. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which may vary from state / province to state / province. This warranty is completely transferable to any and all future owners of this product, provided no alterations have been made to the product.

AUSTRALIA: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

#### **HOW TO SUBMIT YOUR REGISTRATION:**

#### Step 1

Fill out this information for your own records.

#### Step 2

Go online to teeter.com to register your warranty.

Date of Purchase		
Product & Model		
Dealer Name		
Carial Na Jacobsol a	on the back of the Table Bed	

If you are unable to go online, you can request a warranty card to be mailed to you by calling Customer Service at 800-847-0143.

Please DO NOT mail this to Teeter.



Distributed by:
Costco Wholesale Corporation
P.O. Box 34535
Seattle, WA 98124-1535
USA
1-800-774-2678
www.costco.com

Costco Wholesale UK Ltd / Costco Online UK Ltd Hartspring Lane Watford, Herts WD25 8JS United Kingdom 01923 213113

Costco Wholesale Japan Ltd. 3-1-4 Ikegami-Shincho Kawasaki-ku, Kawasaki-shi, Kanagawa 210-0832 Japan 0570-032600 www.costco.co.jp Costco Wholesale Canada Ltd.\*
415 W. Hunt Club Road
Ottawa, Ontario
K2E 1C5, Canada
1-800-463-3783
www.costco.ca
\* faisant affaire au Québec sous

le nom les Entrepôts Costco

Costco Wholesale Spain S.L.U. Poligono Empresarial Los Gavilanes C/ Agustín de Betancourt, 17 28906 Getafe (Madrid) España NIF: B86509460 900 111 155 www.costco.es

Costco Wholesale Korea, Ltd. 40, Iljik-ro Gwangmyeong-si Gyeonggi-do, 14347, Korea 1899-9900 www.costco.co.kr Importado por:
Importadora Primex S.A. de C.V.
Blvd. Magnocentro No. 4
San Fernando La Herradura
Huixquilucan, Estado de México
C.P. 52765
RFC: IPR-930907-S70
(55)-5246-5500
www.costco.com.mx

Costco Wholesale Iceland ehf. Kauptún 3-7, 210 Gardabaer Iceland www.costco.is Costco Wholesale Australia Pty Ltd 17-21 Parramatta Road Lidcombe NSW 2141 Australia www.costco.com.au

> Costco France 1 avenue de Bréhat 91140 Villebon-sur-Yvette France 01 80 45 01 10 www.costco.fr

Costco President Taiwan No. 656 Chung-Hwa 5th R Kaohsiung, Taiwan Company Tax ID: 969727 449-9909 or 02-449-9909 (if o www.costco.com.tw





If you have any trouble assembling the equipment, or questions about its use, please contact customer service.

USA: 800-847-0143 or info@teeter.com (English language) International: info@teeterintl.com

Hours of Operation (Monday - Friday 8am - 4pm PST)

USA: Teeter 9713 233rd Avenue East Bonney Lake, WA 98391 Toll Free: 800-847-0143 Fax: 800-847-0188 teeter.com | info@teeter.com International: Teeter International, Ltd.
Gor-Ray House
758 Great Cambridge Rd
Enfield
Middlesex ENI 3GN
United Kingdom

teeterintl.com | info@teeterintl.com





Any modification to this device will void the UL Listing



Medical Device Safety Service GmbH Schiffgraben 41 30175 Hannover Germany Tel. +49 511 62628630



Teeter Decompression Devices are multiple user, reusable devices for home use, intended to provide traction to the spine while stretching the para-spinal muscles and soft tissues.

The devices provide non-powered traction and are meant for use by adults.

Use of the Teeter Decompression Devices is indicated for the following conditions: back pain, muscle tension, degenerative disc disease, spinal degenerative joint disease, spinal stenosis, herniated disc, spinal curvature due to tight muscles, sciatica, muscle spasm, and facet syndrome.